

Read all Directions Before Using Freeze 'n Clear Skin Clinic[™]

Table of Contents:

- A. The Freeze 'n Clear Skin Clinic[™] kit consists of: E. WARNINGS
 B. When to use Freeze 'n Clear Skin Clinic[™] F. Directions for Use
 C. How does Freeze 'n Clear Skin Clinic[™] work? G. Cautions!
 D. When not to use Freeze 'n Clear Skin Clinic[™]

A. The Freeze 'n Clear Skin Clinic[™] kit consists of:

1. An aerosol canister with a mixture of dimethyl ether, propane, and isobutane. The mixture is flammable but not harmful to the environment.
2. 16 applicators for 16 treatments with Freeze 'n Clear Skin Clinic[™]. 12 medium applicators for medium and large warts, and 4 small applicators for smaller warts.
3. This brochure tells all about the product, how it works and how Freeze 'n Clear Skin Clinic[™] can be used to remove warts.

B. When to use Freeze 'n Clear Skin Clinic[™]

Freeze 'n Clear Skin Clinic[™] is intended for the removal of common warts and plantar warts (on the bottom of the foot). Common warts are non-cancerous growths that consist of a hard core of skin. They can be round or irregular in shape with flat or raised surfaces. Their color can be variations of brown, white or gray or brown. Typically, common warts grow around the nails, on the fingers and on the backs of hands, but can also be found on the knees and face, most often where the skin has been previously broken. You can recognize a common wart by its rough, "cauliflower"-like surface.

You can recognize a plantar wart by its location only on the bottom of the foot, its tenderness, and identified by the way it interrupts the normal pattern of the skin. Plantar warts are usually larger than common warts and are flatter because of the pressure put on the bottom of the foot. Most plantar warts don't stick up above the surface of the skin. They are often confused with calluses and can exist with them.

If you are not sure that the growth on your skin is a wart, see your doctor before you use Freeze 'n Clear Skin Clinic[™]. If you treat a skin condition as a common or plantar wart with Freeze 'n Clear Skin Clinic[™] and it is not a wart, you delay proper treatment. This delay may lead to a more serious skin condition or disease.

C. How does Freeze 'n Clear Skin Clinic[™] work?



Common Wart

Freeze 'n Clear Skin Clinic[™] freezes warts on the spot. As a result of freezing, a blister may form (sometimes containing blood) within a few days. After treatment, the treated wart falls off and new skin grows in its place. This process could take up to two weeks. For **common warts**, usually only 1 treatment is necessary. In persistent cases, repeat the treatment in 2 weeks. Talk to your doctor if the wart persists after 4 treatments.



Plantar Wart

Plantar warts usually do not fall off as readily as common warts and may require up to 4 treatments with 2 week intervals between treatments. Talk to your doctor if the plantar wart persists after 4 treatments.

Do not use Freeze 'n Clear Skin Clinic[™] to treat the same common or plantar wart for more than 4 times total at two week intervals. If the problem does not resolve after 4 treatments, tell your doctor about your problem because the growth may be something other than a wart.

D. When not to use Freeze 'n Clear Skin Clinic[™]

- **Do not use Freeze 'n Clear Skin Clinic[™] on things that are not warts**, such as moles, birthmarks, skin tags, dark-colored, hairy or otherwise unusual looking skin conditions that resemble warts. If you treat a skin condition that is not a wart, you delay proper treatment. This delay may lead to a more serious skin condition or disease, and could damage healthy skin or nerves.
- **Do not use Freeze 'n Clear Skin Clinic[™] on areas of thin skin**, such as your face, armpits, chin, breasts or buttocks. Use on these areas may cause burns and permanent scarring of the skin or blindness.
- **Do not use Freeze 'n Clear Skin Clinic[™] on genital warts, mucous membranes**, such as inside your mouth, nose, anus, genitals, lips, ears or near eyes or on skin conditions with signs of swelling or redness. Use on these areas may cause delayed healing, burns and permanent scarring of the skin.
- **Do not use on children younger than 4 years old** because it is uncertain how Freeze 'n Clear Skin Clinic[™] will affect the child's skin. Use on young children may cause burns and permanent scarring of their skin.
- **Do not use if you are diabetic**, have poor blood circulation or have been diagnosed with blood conditions affected by extreme cold. Freeze 'n Clear Skin Clinic[™] works by freezing the wart. People with diabetes will find that the treated area may not heal as well or completely.
- **Do not use if you are pregnant or breastfeeding.** It is uncertain how Freeze 'n Clear Skin Clinic[™] may affect patients under these conditions.

E. WARNINGS

- **Use only as directed.** Misuse of Freeze 'n Clear Skin Clinic[™] may result in burns and permanent scarring of healthy tissue or blindness.
- **Keep out of the reach of children.** Failure to follow instructions may result in serious injury or harm. Contents are hazardous if swallowed, breathed or sprayed on body parts other than warts.
- Freeze 'n Clear Skin Clinic[™] is toxic if taken internally. **Do Not Put It In Your Mouth Or Nose.** If you get it in your mouth or nose, get medical help or contact a poison control center immediately.

- Do not use on warts in inaccessible or hard-to-see places. If Freeze 'n Clear Skin Clinic[™] is mistakenly applied directly to skin or if it is used on conditions that are not warts, burns and permanent scarring of the skin may result.

Do not use Freeze 'n Clear Skin Clinic[™] in combination with any other method of wart removal. It is uncertain how the combination may affect the skin. Combination usage may result in burns and permanent scarring of the skin.

- **Always use the applicator supplied with the product when applying Freeze 'n Clear Skin Clinic[™].** Do not spray gas directly onto skin, as burns and permanent scarring of the skin may result. Freeze 'n Clear Skin Clinic[™] may not be effective if used in combination with other applicators.
- Use one applicator for each common or plantar wart. Do not reuse the applicator to ensure that any possibility of re-infection with the wart virus does not occur.
- If there are multiple warts on the foot or hand, treat only one wart at a time. Treat each remaining wart separately with a 2 week interval between treatments. This minimizes any chance of "over freezing", which can result in burns and permanent scarring of healthy tissue.
- Hold the applicator on the wart for no more than 40 seconds. Overexposure can lead to freezing too deeply with damage to the skin, resulting in scars and/or nerve damage.
- Discard applicator immediately after use to reduce the occurrence of re-infection with the wart virus. No special method of disposal is required.
- Freezing the fingers/foot/toes too deeply can lead to local tissue death. To avoid this, treat only one surface of the foot at a time to avoid freezing arteries or veins. For warts on the foot, wiggle and bend the toes to improve their blood circulation while treating areas near them. For warts on the hands, wiggle and bend the fingers to improve blood circulation while treating areas near them.
- Depigmentation (a temporary loss of normal skin color) sometimes occurs when freezing treatments are utilized. Normal skin color will return gradually.

F. Directions For Use


PLANTAR WARTS
 Read All Directions Before Using
 Freeze 'n Clear Skin Clinic[™]
DO NOT SPRAY DIRECTLY ONTO SKIN! • DO NOT APPLY THE APPLICATOR IMMEDIATELY!

Getting ready


1. Soak the foot with the plantar wart in warm water for 5 minutes. Dry.



2. In your kit you will find all components as shown in drawing. You will need a clock or household timer that counts in seconds as a way to time your treatment.

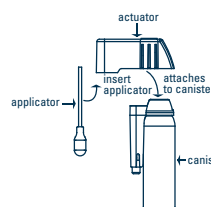


3. Hold an Freeze 'n Clear Skin Clinic[™] applicator by the plastic stick.
 - Attach the applicator stick to the actuator by firmly twisting it into the opening on bottom of the actuator as shown.

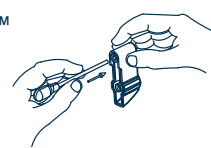


4. Place the canister on a table or other sturdy surface. This makes it easier to press the actuator. **Do not hold the spray can near your face, over parts of your body or clothing!**

5. Insert applicator and actuator as shown.
 - Keeping the canister upright, press the actuator all the way down for 3 seconds or until you have saturated the applicator.
 - You will hear a hissing sound.
 - **Do not let any liquid make contact with your skin.**

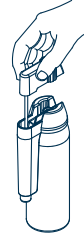

6. Remove the applicator and actuator from the shield and **wait 15 seconds before applying to the wart.** This permits the applicator tip to reach the correct temperature.

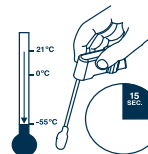

Treating the Wart

7. Expose the plantar wart to be frozen facing upward.



8. After waiting 15 seconds and while keeping the applicator pointing straight down, **lightly** place the tip of the applicator on the center of the plantar wart to be frozen. Apply the applicator to the treated area continuously for 40 seconds and no longer. Do not dab. It is normal to feel an aching, stinging or painful sensation during treatment and for a short time afterwards.



9. Discard the foam applicator after a single use to ensure that any possibility of re-infection with the wart virus does not occur. No special method of disposal is required.

10. **If the wart has still not fallen off or disappeared in two (2) weeks, you may treat the wart again, up to 4 treatment cycles.**

If the wart has still not disappeared, stop using the product and ask a doctor about the wart.

Follow-up treatment

1. As a result of freezing, a blister may form and may contain blood. If this occurs, discontinue treatment. New skin will grow in the place of the plantar wart. This may take up to two weeks.

2. To help you avoid infection:
 - Keep the treated area clean.
 - Do not pick at, or scratch, the treated area.
 - Protect blisters, if necessary, with a bandage; do not puncture blisters. Swimming and showering are permitted.

Talk to your doctor if you have burning, aching or stinging sensations that do not go away or if you think the treated area is getting infected.

3. **If the wart has still not fallen off or disappeared in two (2) weeks, you may treat the wart again, up to 4 treatment cycles.**

If the wart has still not disappeared, stop using the product and ask a doctor about the wart.

G. Cautions!

- **Extremely Flammable.** Keep away from fire or flame. Do not smoke while using Freeze 'n Clear Skin Clinic[™]. Do not expose to heat or store at temperature above 120°F. Store at room temperature away from heat. Exposure to heat may cause the canister to explode, causing serious injury.
- **Contents under Pressure.** Do not puncture or incinerate canister, even after use. This may cause the canister to explode, causing serious injury.
- **Avoid Contact with Eyes.** Contact with eyes may cause blindness. Do not rub or touch eyes with hands if Freeze 'n Clear Skin Clinic[™] has been applied to them. If product gets into eyes, flush with water for 15 minutes and get medical help right away.
- **Avoid Inhaling and Use Only in Well-Ventilated Areas.** It is uncertain how the vapor/spray will affect the lungs. Breathing the vapor/spray is hazardous.
- **Do Not Eat.** Contents are toxic and may cause serious internal damage. If eaten by accident, get medical help or contact a poison control center immediately.
- **For External Use Only.**

Thank you for buying

Freeze 'n Clear
Skin Clinic[™]
 ADVANCED
Wart Remover

If you have any questions, please call consumer affairs at 1-866-758-1931 and a representative will be happy to help you.